# Grab Bar • Heavy Duty

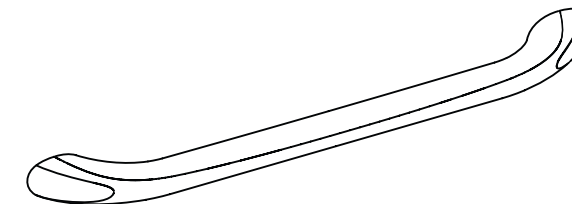
INSTALLATION, OPERATING  
AND CARE INSTRUCTIONS

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# Grab Bar • Heavy Duty

## INSTALLATION, OPERATING AND CARE INSTRUCTIONS

### SAFETY WARNINGS:

- **DO NOT** install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these Warnings and Instructions, contact a health care professional, dealer or technical personnel for assistance before attempting to install this equipment - otherwise, injury may occur.
- After ANY adjustments, repair or service and BEFORE use, make sure that all attaching hardware is tightened **securely**.
- Grab Bars add security for bath and commode areas if attached as instructed. Location, size, and angle of bars should be determined by a qualified professional to provide optimum safety and function.
- Use **EXTREME** caution on wet tub or floor surfaces.
- Users with limited physical capabilities should be supervised or assisted in bath and commode areas, even when using Grab Bars.
- This Grab Bar can provide support and increased stability for an individual weighing up to **500 pounds**. Grab Bars are NOT designed to support the total weight of an individual. **Use the bar for assistance ONLY.**
- Be sure Grab Bars are correctly and securely installed. NEVER use Grab Bars if any looseness is noted.

### TOOLS REQUIRED:

- Safety Glasses
- Pencil
- 3/8-inch Power Drill
- 3/16-inch Drill Bit (to drill pilot hole for installation of Lag screw into wood wall stud)
- Phillips Screw Driver
- 5/16-inch Carbide Tip Drill Bit (if drilling through ceramic tile)
- 7/16-inch Hex Socket with 3/8-inch Drive Ratchet
- Stud Finder

### PARTS SUPPLIED:

- Two (2) 1/4-inch x 3-inch Hex Head Lag Screws
- Two (2) 1/4-inch Flat Nylon Washers

### IMPORTANT:

The recommended mounting method for grab bars is to mount **BOTH** ends of the bar into **WOOD** wall studs. For help locating wood wall studs, please visit our website at [www.libertyhardware.com](http://www.libertyhardware.com).

### BEFORE INSTALLING GRAB BAR:

1. Remove the Grab Bar and parts from the package.
2. Determine where you want to install the grab bar. **Note: The soft grip should always face the palm of your hand (see Figure 1).**
3. Remove the snap cover from the bar by pulling straight out from the side (see Figure 1). **Note: Each snap cover is specific to a side. On the back of each snap cover, RH for right side or LH for left side is shown.**
4. With a pencil, mark mounting holes at desired location on the wall. If installing over ceramic tiles, place masking tape on the mounting area before marking the hole location.

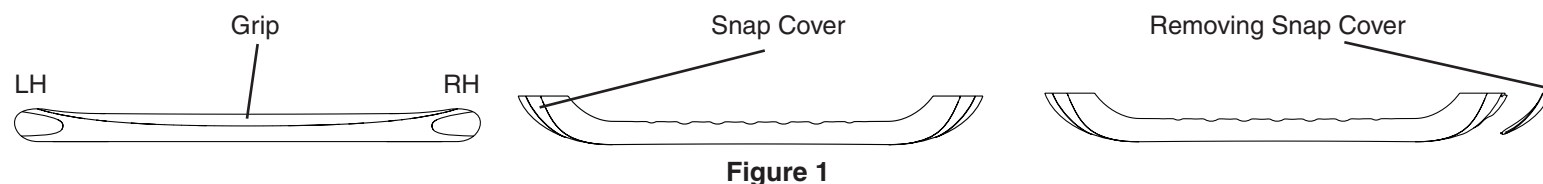
Figure 1

## BOTH ends of grab bar into WOOD wall studs

### WARNING

**GRAB BAR MAY NOT PROVIDE DESIGNED AMOUNT OF SUPPORT UNLESS THESE INSTALLATION INSTRUCTIONS ARE STRICTLY FOLLOWED.**

For proper installation, both ends of Grab Bar MUST be positioned over a wall stud. For solid support, both fasteners MUST go through the wallboard or tile and be tightly secured into the wall stud. Grab Bar may be installed vertically, horizontally or at an angle, so the mounting screw on both ends of the Grab Bar is secured into a wall stud (See Figures 2 & 3). Locate the wall studs behind the wall before beginning installation (a stud finder can be obtained from a hardware store).

### WARNING

**THE FOLLOWING STEPS REQUIRE DRILLING INTO THE WALL.** When drilling into the wall, exercise care to avoid any electrical wiring or plumbing that may be located behind the wall. Damaged electrical wiring can cause electrical shock and/or fire. Since older homes do not always fall in line with current housing codes and requirements, know where internal wall wiring is located so that no wires will interfere with your installation. **PROPER INSTALLATION IS EXTREMELY IMPORTANT. IF IN DOUBT, INSTALLATION SHOULD BE DONE BY A QUALIFIED PROFESSIONAL.**

### TO INSTALL INTO WOOD WALL STUDS:

5. Put on safety glasses before starting.
6. Use the 3/8-inch power drill and 3/16-inch drill bit to drill a hole into the wall and wood wall studs at the marked position. **NOTE: If installing Grab Bar over ceramic tiles, the 5/16-inch carbide tip drill bit will be needed for drilling through the tile. Drill clearance hole through the ceramic tile by using a 5/16-inch carbide tip drill bit. Once a hole has been drilled through the tile, use the 3/16-inch drill bit to drill a pilot hole in the wood wall stud.**
7. Place the Grab Bar on the wall, aligning mounting hole with the hole in the wall.
8. Insert the Lag screw and nylon washer provided into one end of the Grab Bar. Screw into wall and wall stud (DO NOT COMPLETELY TIGHTEN).
9. For opposite end of the Grab Bar, rotate to align with the wood wall stud on other end.
10. Hold the bar at the aligned location and mark the drill hole location on the wall with a pencil.
11. Repeat Step 6.
12. Insert the Lag screw and nylon washer provided into the end of the Grab Bar. Screw into wall and wall stud (see Figure 3).
13. Securely tighten both ends using 7/16-inch Hex socket with 3/8-inch drive ratchet.
14. Replace both snap covers. Be sure to check the back of each snap cover to ensure you replace it on the correct side.

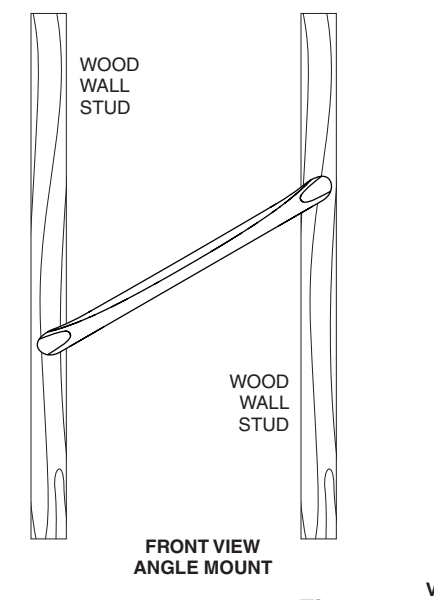
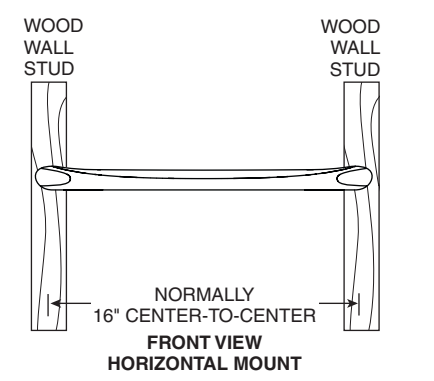
Figure 2

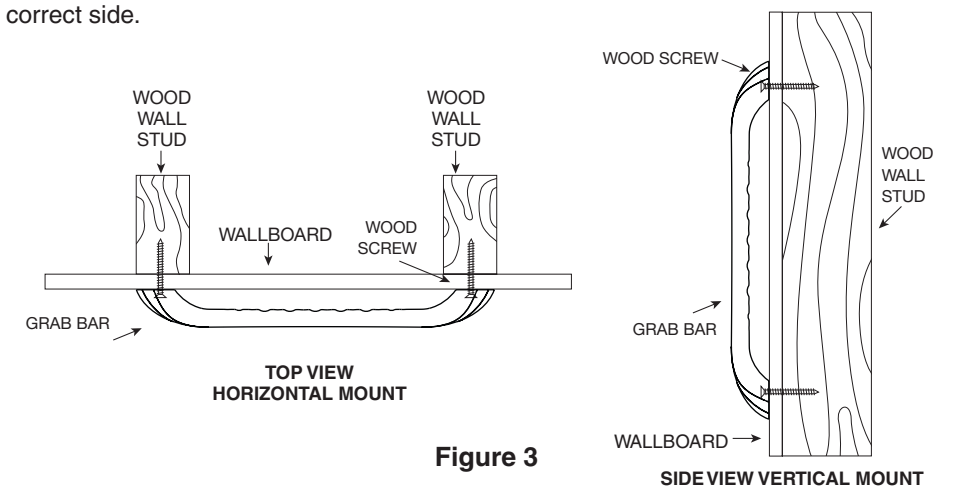
Figure 3

### USING GRAB BAR:

1. Read, understand and follow the information in the **SAFETY WARNINGS** section of this instruction booklet. Liberty Hardware Manufacturing Corporation shall have no liability for any damage or injury caused by improper installation or use of this Grab Bar.
2. Grasp the Grab Bar when exiting or entering the bathtub or when sitting down or rising from the commode.
3. Use **EXTREME** caution on wet tub or floor surfaces.
4. **DO NOT** put entire weight on Grab Bar. Grab Bar is rated for persons weighing up to **500 pounds** when properly installed.
5. **DO NOT** stand, sit or step on Grab Bar. **DO NOT** use Grab Bar as a step or platform to reach area above the tub. **Use the bar for assistance ONLY.**

### CARE AND MAINTENANCE:

1. Use of mild soap and water or a non-abrasive cleaner is recommended for cleaning.
2. Periodically inspect the Grab Bar for damage or looseness. **DO NOT** use if any damage or looseness exists. Contact your authorized dealer for assistance.
3. **SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE.**

FOR RECOMMENDATIONS ON GRAB BAR PLACEMENT, PLEASE VISIT OUR WEBSITE AT [WWW.LIBERTYHARDWARE.COM](http://WWW.LIBERTYHARDWARE.COM)

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