Locking Elevated Toilet Seat Instructions

INSTALLATION, OPERATING AND CARE INSTRUCTIONS:

CAUTION AND SAFETY WARNINGS:

Before each use, check that the unit is fully engaged and that the Latch Lever is in the **locked** position. For proper function, this elevated toilet seat must sit evenly on the porcelain toilet bowl rim. This product is designed to fit most toilet designs that have an industry standard seat mounting hole pattern that measures 5-1/2 inches center-to-center. Do not use if installation causes stress on any porcelain components of the toilet tank or rim or if for any reason the product does not feel stable on your particular model toilet. This product has a maximum user weight of **250 pounds**. NEVER MAKE SUBSTITUTIONS OR MODIFY THIS PRODUCT IN ANY MANNER. Do not jump or stand on the seat. Liberty Hardware® Manufacturing Corporation assumes no liability for any damage or injury caused by the improper installation or use of this product. It is recommended that you consult a health care professional for instructions specific to your needs. Physically challenged users should be supervised or assisted whenever using this product. Liberty Hardware® manufactures two locking elevated toilet seats, a round model and an elongated model. Make sure you have the correct version that best fits your particular toilet design.

TOOLS REQUIRED:


- #2 Phillips Head Screwdriver
- 1/4-inch Straight Blade Screwdriver (to remove existing toilet seat)
- Pliers (to remove existing toilet seat)

Note: depending on the hardware used on your existing toilet seat, other tools may be required. Inspect your model to determine actual tools needed.

PARTS SUPPLIED:

- Locking Latch Lever Bracket
- Elevated Seat with Latch Receiver Bracket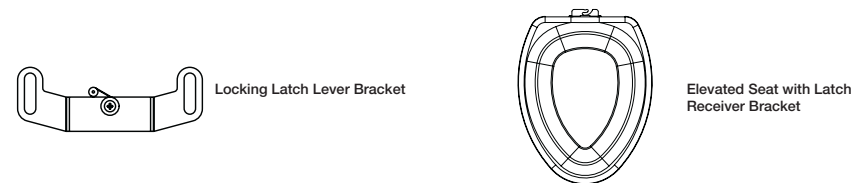

INSTALLATION INSTRUCTIONS:

1. Remove existing toilet seat using tools appropriate for your seat model.
2. Place Elevated Seat with Latch Receiver Bracket onto the porcelain bowl so that the receiver bracket is facing toward the toilet tank (see Figure 1).
3. Adjust the receiver bracket using a #2 Phillips head screwdriver so that the underside of the bracket makes contact and sits flush to the porcelain rim (see Figure 2).
4. Tighten screws after making the proper adjustment. Remove Elevated Seat with Latch Receiver Bracket.
5. Place the Locking Latch Lever Bracket over the porcelain bowl mounting holes to make sure the bracket properly aligns with them.
6. Position the Locking Latch Lever Bracket so the cam lever is facing toward the tank. Move the cam lever into the unlocked position (see Figure 1).
7. Slide the Elevated Seat with Latch Receiver Bracket into the Locking Latch Lever Bracket. Move the cam lever to the locked position (see Figure 3).
8. Position the existing toilet seat over the Locking Latch Lever Bracket and loosely install the mounting hardware. Make sure the bolts pass through the holes in both the existing toilet seat and the Locking Latch Lever Bracket.
9. With the toilet seat mounting hardware loosely installed, pull the elevated toilet seat forward (away from tank) as far as the assembly will allow.
10. Tighten the toilet seat mounting hardware to secure (do not over tighten).

REMOVING THE LOCKING ELEVATED TOILET SEAT:

1. Move the cam lever on the Locking Latch Lever Bracket into the unlock position (see Figure 1).
2. Lift up the front of the Elevated Toilet Seat and pull forward (away from tank).

CLEANING INSTRUCTIONS:

To clean this product, use a non-abrasive mild detergent with warm water and dry.

Do not use abrasive cleaners or bleach.

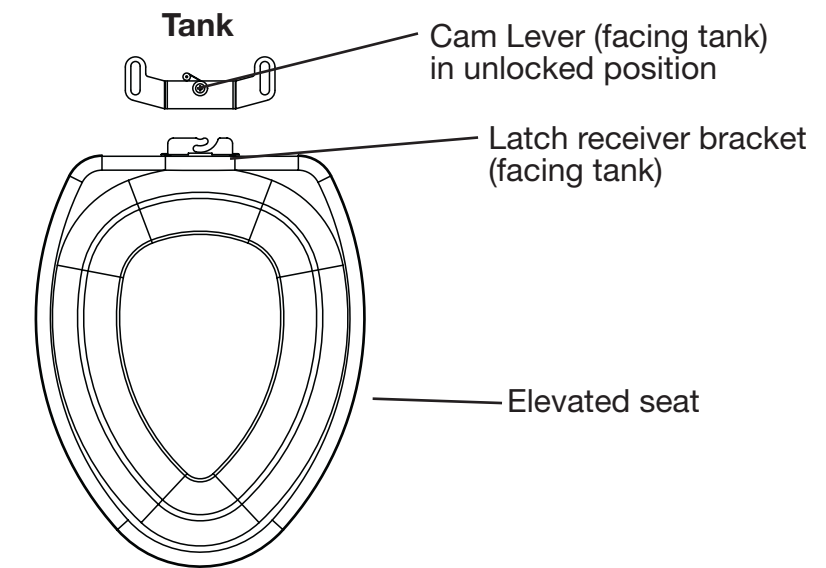
Figure 1

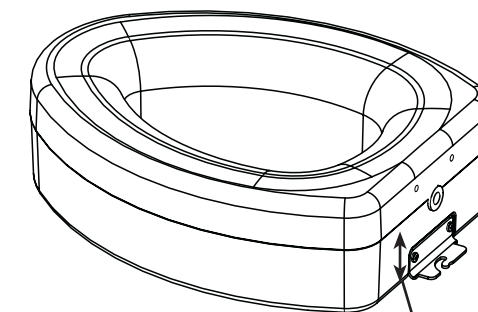
Figure 2 Adjust for proper fit

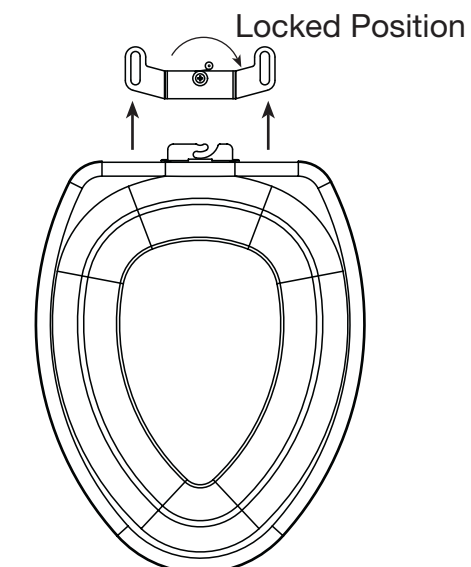
Figure 3

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